

How to test your blood sugar using the Accu-Chek® Aviva Plus system.

1

After washing and drying your hands, pull the cap off your Accu-Chek Softclix device. Insert a lancet and twist off its cover.

2

Replace the cap and press the button at the end of the device to prime it. The release button on the side will turn yellow.

3

Remove one test strip from the vial and insert it into the meter until the meter beeps.

4

Place the lancing device alongside your finger, and press the yellow button to gently prick your finger.

5

Touch the small blood drop with the edge of the test strip's yellow stripe. Do not put blood on the *top* of the test strip, only the *yellow edge* of it.

6

The meter will beep and flash  when there is enough blood.

7

A result will appear in 5 seconds to let you know your blood sugar level.

8

Remove the lancing device cap. Point the lancet away from you over a puncture-proof container and slide out the ejector to discard the lancet.

Note: Meter and lancing device should never be used by more than one person.

QUICK REFERENCE GUIDE

Possible error messages and what to do about them:



E-1

Test strip damaged or not properly inserted. Remove and reinsert the strip, or replace if damaged.



E-4

Not enough blood was drawn into the test strip. Throw away the strip and repeat the test.



E-6

Blood was applied to the test strip before the flashing drop symbol appeared. Throw away the strip and repeat the test.



E-9

Battery is almost out of power. Change battery.

For a full list of error messages please consult your Owner's Booklet that came with your meter.

If you have additional questions or want more information about using your Accu-Chek Aviva Plus system, please contact us:

800-858-8072

accu-chek.com/aviva



ACCU-CHEK, ACCU-CHEK AVIVA, and SOFTCLIX are trademarks of Roche.
© 2015 Roche. 305-54694-0715
07764782001



ACCU-CHEK®